

Want Faster Results? Here are some reminders!

Use the Right Device:

Always use the correct collection tubes and containers provided for each specific test.

 <p>White Top</p> <p>USE FOR: Respiratory, Genitourinary, and all Specialty menus <i>Transfer no more than 1 mL of urine</i></p>	 <p>Senior (65+)</p> <p>USE FOR: Vaginitis, CGT (Vaginal only), CG (Vaginal, Rectal, Oral) EMR CODE: VAGGR, CGTGY, CGGY <i>Do not use lubricants or other products containing substances such as carbomers</i></p>	 <p>Senior (65+)</p> <p>USE FOR: CGT, CG (Urine only) EMR CODE: CGTGY, CGGY <i>Transfer 2 mL of urine</i></p>	 <p>Senior (65+)</p> <p>USE FOR: Urine Culture EMR CODE: UCULT</p>
---	--	--	--

Sample Ordering Reminders

Order Correctly

- ▶ One sample tube per menu ordered
- ▶ Click send on the order

Label Clearly


- ▶ Patient first and last name + date of birth
- ▶ Include collection site on all tubes

Seal and Store

- ▶ Secure lid tightly
- ▶ Seal specimen bag
- ▶ Include necessary paperwork
- ▶ PCR samples are stable at room temperature for 7 days, culture sample for 48 hours.

Sample Pick-Up

- ▶ Place all samples in one shipping bag
- ▶ Seal bag completely

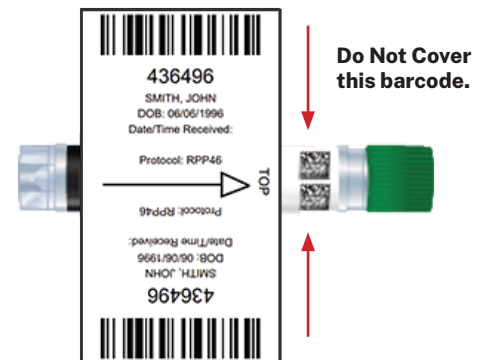

Do NOT apply labels or markings across the cap or over the barcode at the top of the product label. The label must wrap fully around the tube and overlap itself, wrapped tightly and smoothed securely onto the tube with no loose edge, tail or wing exposed.

