

Want Faster Results? Here are some reminders!

Use the Right Device:

Always use the correct collection tubes and containers provided for each specific test.



White Top

USE FOR:

Respiratory, Genitourinary,
and all Specialty menus

Sample Ordering Reminders

Order Correctly

- ▶ One sample tube per menu ordered
- ▶ Click send on the order

Label Clearly


- ▶ Patient first and last name + date of birth
- ▶ Include collection site on all tubes

Seal and Store

- ▶ Secure lid tightly
- ▶ Seal specimen bag
- ▶ Include necessary paperwork
- ▶ Samples are stable at room temperature for 7 days

Sample Pick-Up

- ▶ Place all samples in one shipping bag
- ▶ Seal bag completely

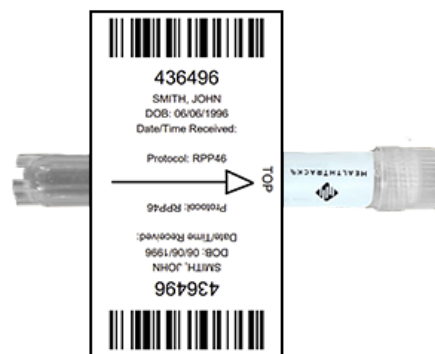

Do NOT apply labels or markings across the cap. The label must wrap fully around the tube and overlap itself, wrapped tightly and smoothed securely onto the tube with no loose edge, tail or wing exposed.

