

## Want Faster Results? Here are some reminders!

### Use the Right Device:

Always use the correct collection tubes and containers provided for each specific test.



**White Top**

**USE FOR:**  
Respiratory, Genitourinary,  
and all Specialty menus  
*Transfer no more than 1 mL of urine*



**Senior (65+)**

**USE FOR:**  
Vaginitis, CGT (Vaginal only),  
CG (Vaginal, Rectal, Oral)  
**EMR CODE:**  
VAGGR, CGTGY, CGGY  
*Do not use lubricants or other products  
containing substances such as carbomers*



**Senior (65+)**

**USE FOR:**  
CGT, CG (Urine only)  
**EMR CODE:** CGTGY, CGGY  
*Transfer 2 mL of urine*



**Senior (65+)**

**USE FOR:**  
Urine Culture  
**EMR CODE:** UCULT

## Sample Ordering Reminders

### Order Correctly

- ▶ One sample tube per menu ordered
- ▶ Click send on the order

### Label Clearly


- ▶ Patient first and last name + date of birth
- ▶ Include collection site on all tubes

### Seal and Store

- ▶ Secure lid tightly
- ▶ Seal specimen bag
- ▶ Include necessary paperwork
- ▶ PCR samples are stable at room temperature for 7 days, culture sample for 48 hours.

### Sample Pick-Up

- ▶ Place all samples in one shipping bag
- ▶ Seal bag completely

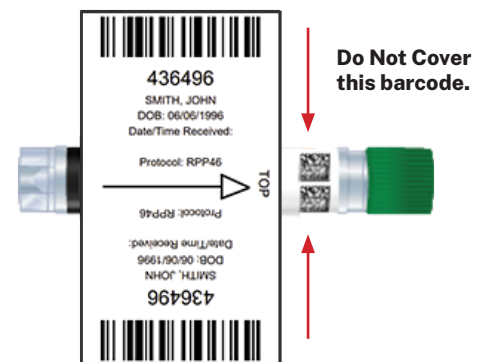

**Do NOT apply labels or markings across the cap or over the barcode at the top of the product label.** The label must wrap fully around the tube and overlap itself, wrapped tightly and smoothed securely onto the tube with no loose edge, tail or wing exposed.

